



WIP'ANIOS Excel

The choice of **safety** into a **suitable format**

Make your choice!



CLEANSING DISINFECTANT WIPES



Wip'Anios Excel Travel 25

- 1 24 packets 25 wipes each
Size : 130 x 210 mm
Texture : 100% viscose
Ref. 2617.466

Wip'Anios Excel 60

- 2 12 packets 60 wipes each
Size : 240 x 300 mm
Texture : 40% viscose 60% polyester
Ref. 2446.492

Wip'Anios Excel 50 et 100

- 3 12 packets 50 wipes each
Size : 180 x 200 mm
Texture : 100% viscose
Ref. 2446.424

- 4 6 packets 100 wipes each
Size : 180 x 200 mm
Texture : 100% viscose
Ref. 2446.655

Wip'Anios Excel King Size

- 5 1 bucket containing 1 refill of 350 wipes
Size : 200 x 360 mm
Texture : 70% viscose 30% polyester
Ref. 2446.672

- 6 2 refills of 350 wipes
Size : 200 x 360 mm
Texture : 70% viscose 30% polyester
Ref. 2446.673



Pavé du Moulin 59260 Lille-Hellemmes - FRANCE
Tél. +33 3 20 67 67 67 Fax. +33 3 20 05 27 48
www.anios.com

EN2016-08 - Non contractual pictures. Dangerous- respect the precautions of use. Medical Devices class I/II. Use biocides with caution. Before using, read the label and information relative to the product. Biocidal product.



WIP'ANIOS Excel

WIP'ANIOS EXCEL wipes combine cleaning and disinfection in a single step making it possible to ensure **macroscopic and microscopic hygiene of high surfaces and medical devices**, at any time, in any environment (operating room, dialysis department, environment near the patients, medical office, interiors of medical vehicles, etc.).

✓ Cleansing disinfectant wipes

- Impregnation solution: Surfa'safe Premium.
- Efficiency ensured until the expiry date.
- Complies with EN 16615 phase 2/step 2.

✓ Ready to use

- Uniform and controlled impregnation, no dilution in qualitative water which is not controlled.
- Assured traceability.

✓ Suitable for all types of surfaces

- In particular, haemodialysis generators, including those that may come into contact with foodstuffs (Group 1 - PT2 & PT4).
- Class IIb non-immersible medical devices.
- Wiping endoscope sheaths in preliminary treatment.
- Wide compatibility with a wide range of materials.

✓ Formats suited to different uses

- Available in packets and buckets.
- 4 different sizes.

For the safety of your patients, operators and equipment.



SAFETY GUARANTEED



MULTI-USE



ENSURED VERSATILITY

SAFETY GUARANTEED



FOR THE USER

The impregnation solution is formulated :

- Without CMR substances
- Alcohol free
- Fragrance free

for **optimal use tolerance and comfort**.

FOR EQUIPMENT

The wipes are **impregnated with a Surfa'safe Premium solution**: its neutral pH and lack of amphoteric surfactants ensure maximum compatibility with a wide range of materials (metals, polymers, polycarbonate, aluminium, etc.).

FOR THE ENVIRONMENT

The **100% biodegradable viscose** of WIP'ANIOS EXCEL (50 or 100) and WIP'ANIOS Excel Travel wipes makes it possible to limit the impact on the environment.

VALIDATED ANTIMICROBIAL EFFICACY

- › Cleansing disinfectant wipes validated according to EN 16615.
- › Bactericide, fungicide and virucide impregnation solution.

ACTIVE AGAINST	STANDARDS	CONTACT TIME
BACTERIAS	EN 13727	2 minutes
	EN16615	2 minutes
	EN 13727 on VRE, A.baumannii βLSE, E.coli OXA 48, E.cloacae OXA 48, K.pneumoniae OXA 48, MRSA	1 minute
	EN 13697 sur ERV, A.baumannii βLSE, E.coli OXA 48, MRSA	2 minutes
YEASTS	EN 13624	2 minutes
	EN16615	2 minutes
MOULDS	EN 1275, EN 13624, EN 13697	20 minutes
VIRUS	EN 14476 : Adenovirus, Norovirus Murin	30 minutes
	PRV, (surrogate of HBV), BVDV (surrogate of HBV), Vaccinia virus, HSV, Rotavirus	1 minute
	Polyomavirus SV40	5 minutes

STANDARD EN 16615

- › The application standard phase 2/step 2 **EN 16615** (May 2015) makes it possible to claim activity combining the action of the wipe and the solution by measuring the **bactericidal and yeasticidal efficacy** of a wipe pre-impregnated with a disinfectant cleansing solution.

Test condition: Quantitative surface test with mechanical action using wipes (4 zone test) in 1 to 5 minutes (or 60 minutes), at a temperature between +4°C and +30°C, diluted in 30°f hard water for concentrated products and in the presence of interfering substances (CP: 0.3g/L; CS: 3g/L albumin + 3mL/L erythrocytes).

A significant reduction must be established to claim the "cleansing disinfectant wipe" status: 5 log reduction for bacterias - 4 log reduction for yeast.

MAKE YOUR CHOICE !

Wip'Anios Excel Travel 25



Size : 130 x 210 mm
Texture : 100% viscose

Wip'Anios Excel 50 ou 100



Size : 180 x 200 mm
Texture : 100% viscose

Wip'Anios Excel 60



Size : 240 x 300 mm
Texture : 40% viscose 60% polyester

Wip'Anios Excel King Size



Size : 200 x 360 mm
Texture : 70% viscose, 30% polyester

- Microbiological quality managed : Radio-sterilized wipes for an optimum safety.



Particularly suitable for haemodialysis generators

TRAVEL SIZE

PERFECT FOR ...

- **Small medical devices** (pressure sensors, blood sugar testers, tensiometres, reflex hammers, conventional ultrasound probes)
- **Wiping endoscopes** preliminary treatment before disinfection
- **Surfaces in the vicinity of patients** (adaptable, bed, stretcher)

PERFECT FOR ...

- **Medical equipment and devices**
- Surfaces requiring wipes in a larger format (external device structures, examination table, dialysis equipment, etc.)