

# **SOLUSCOPE** *SPRINT PT*

Cutting-edge performance  
in Pass-Through



**Soluscope**


CE  
1434

# SOLUSCOPE SPRINT PT

reinvents

# Pass-Through

experience

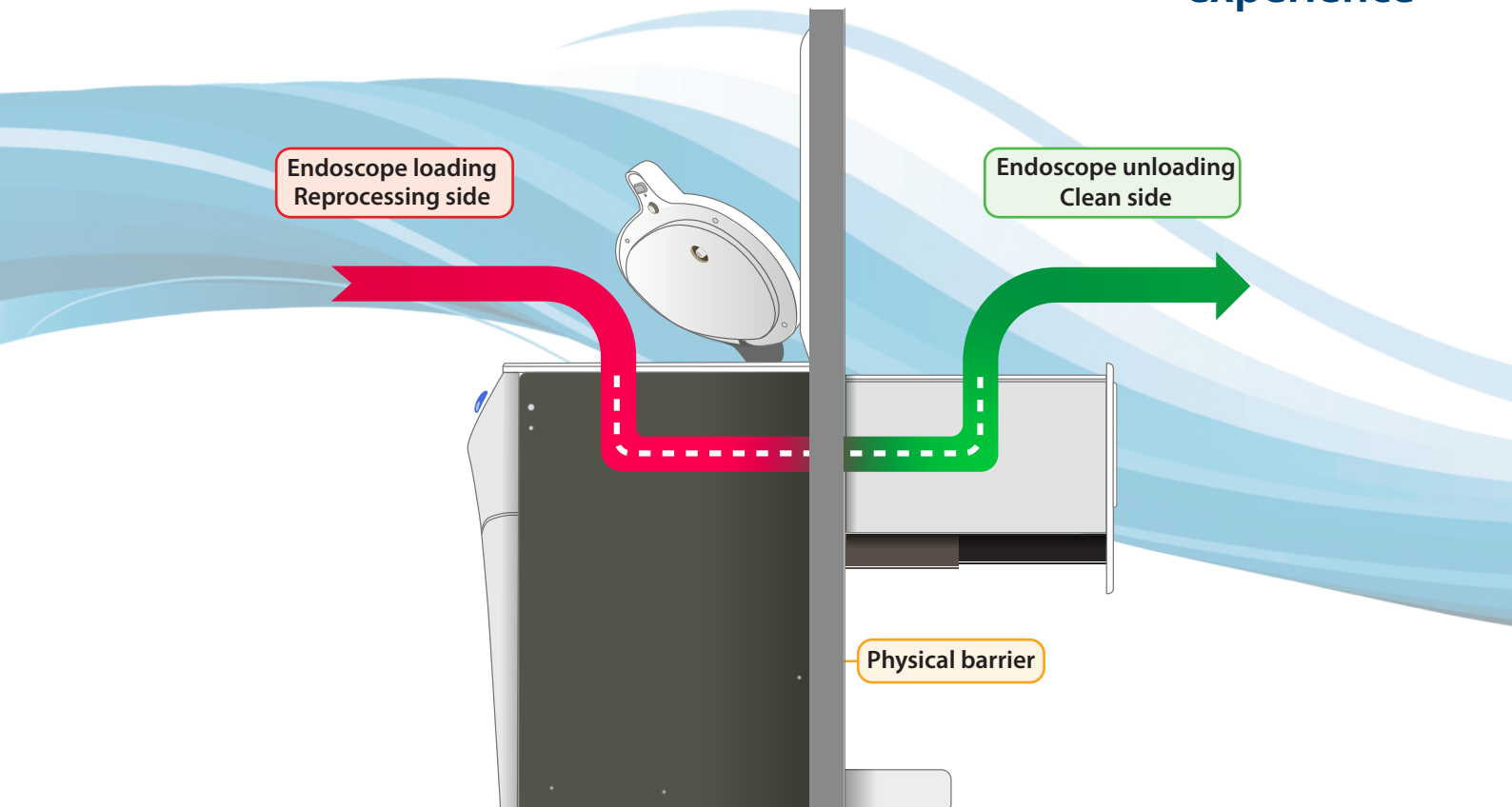

## Serving department efficiency

### “ONE WAY” PROCESS FOR PATIENT SAFETY AND REPROCESSING PERFORMANCE


Separation of reprocessing areas from clean areas is globally recognised as a good practice of AER configuration to reduce the risk of cross contamination. The *Soluscope Sprint PT* contributes to implementing the high-level hygiene practices.

### WORKFLOW OPTIMISATION

The *Soluscope Sprint PT* offers the best time-effectiveness ratio for performing a complete cycle, for any types and brands of flexible endoscopes. Moreover, thanks to a unique maintenance system, the physical barrier is preserved: there is no interruption of your reprocessing activities on other equipment during technical intervention.



CE MARK / ISO 15883 - 1 & 4 compliant

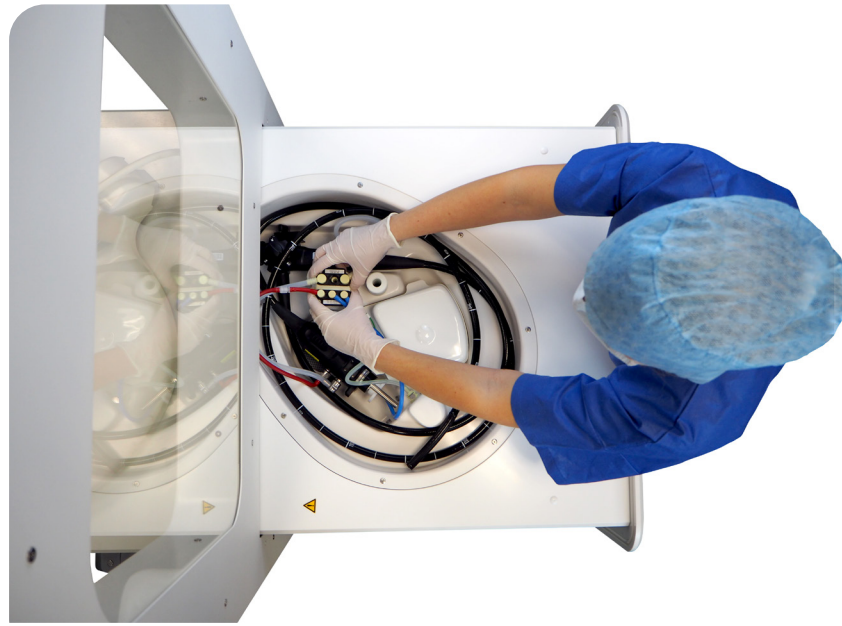

# User-centered design

## *Advanced ergonomics for a better control*

### **SMART, EASY-TO-USE INTERFACES FOR A FAST ADOPTION**

With LED lights flashing on the clean side, the user is continuously kept informed of the status of the machine.

On the reprocessing side, the large touch screen provides fast access to cycle launch and parameter settings.

Furthermore, the window in the wall frame enables easier communication between the 2 areas.

### **EMBEDDED HARDWARE TO GUARANTEE DIGITAL TRACEABILITY**

The *Soluscope Sprint PT* includes barcode readers on both sides for accurate data entry, printer and local data storage.

### **MINIMISED HANDLING TO REDUCE CROSS-CONTAMINATION RISKS**

Dedicated connectors and automatic pass-through drawer provide minimal handling of the endoscope.

### **EASY ACCESS MAINTENANCE AND SERVICE**

Maintenance activities are performed from the reprocessing area, including filters and chemical replacement, which are easily accessible.

automated endoscope reprocessor

when

# Performance

meets Patient Safety



## Proven safety

Proven and tested to achieve the best results from your *Sprint PT*, *Soluscope* range of high-performing chemistry is compatible with all brands of endoscopes.

### SOLUSCOPE P

5% peracetic acid-based stabilised disinfectant. Optimised minimum effective concentration and contact time (5 minutes). Non foaming.

### SOLUSCOPE A

Anticorrosive additive to *Soluscope P* to protect machine and endoscopes.


### SOLUSCOPE CLN+

- Tri-enzymatic neutral cleaner
- Pre-digests organic residues (proteins, carbohydrates, fats)
- Non-foaming
- Efficient against biofilm

# Connectors

Offering compatibility across a range of *Soluscope* equipment, the connection set follows the endoscope journey from reprocessing to drying/storage.

This association allows the connector set to be cleaned and disinfected with every scope, ensuring an effective, safe and proven link between all *Soluscope* equipment and endoscopes.

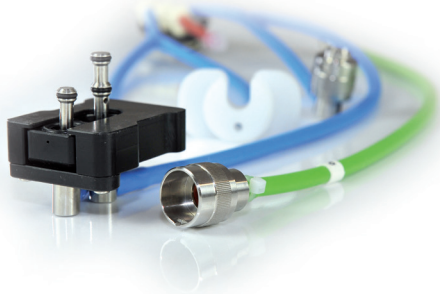

## TECHNICAL SPECIFICATIONS

Capacity per cycle	1 endoscope
Reprocessing cycle time	22 minutes
Disinfection temperature for HLD cycle	40°C
Contact time / disinfection for HLD cycle	5 minutes
Traceability	2 barcode readers - Printer - Memory Card - IT Soluscope (option)
Electrical connection	1 x 220-240 V / 50 Hz with 30 mA differential power breaker
Clean area: drawer depth	65.7 cm
Dimensions reprocessing area (W x D x H)	70 x 57 x 101 cm (lid closed)
Weight (empty)	235 Kg
Weight (wall frame)	50 Kg

Please contact your local *Soluscope* representative for the complete installation specification.

## A complete solution to optimise your organisation

### TRANSPORT

*Soluscope Trolley* provides safe transport of your endoscopes throughout your facility.

These trays, straight from your *Soluscope* Drying & Storage cabinet, feature transparent lids, to keep the scopes safe during transportation.

*Soluscope transport solution* facilitates your daily routine in a safe manner.

### STORAGE

*Soluscope Rack* can store up to 10 trays in a reduced space.

### SERVICES

After-sales service, maintenance and training are designed to get the best of your device.



## Data management solution



The *Sprint PT* is compatible with the global traceability software solution *IT Soluscope* which provides access to your endoscope database in real time for data management.



Soluscope  
REGISTERED TO ISO  
9001:2015 AND 13485:2016  
FILE NUMBER A14025



## SOLUSCOPE COMPLETE SOLUTION

*Soluscope* proposes a complete solution for flexible endoscope reprocessing.

It offers products for every step of the reprocessing cycle.

Each part is certified, easy to use and effective in order to reduce your daily load.



[www.soluscope.com](http://www.soluscope.com)

Soluscope SAS : 100, rue du Fauge - Z.I. Les Paluds - 13400 Aubagne - France

☎ +33 (0)4 91 83 21 22 📠 +33 (0)4 91 83 21 10 ✉ info@soluscope.com

*Soluscope* products are compliant with *European and international standards*, and are *CE marked*. *ISO 9001 version 2015 and ISO 13485 version 2016 certified*, *Soluscope* is continuously committed to satisfying its clients' rigorous demands.

distributed by