**SELECTA
REINIGUNGSPISTOLE**

pat. 1769534

***SELECTA
SPRAY GUN***

pat. 1769534

STOCKERT
INSTRUMENTE GMBH

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SIMPLY INDISPENSABLE IN THE CLINIC, LABORATORY, OR MEDICAL PRACTICE

The Selecta Spray Gun is the ideal instrument for cleaning laboratory glassware, pipettes, syringes, catheters, cuvettes, endoscopes, cystoscopes, cannulae and a variety of other instruments and articles used in clinics, laboratories, and medical practices – in other words, everywhere where highest cleanliness and hygiene are mandatory.

The Selecta Spray Gun uses either water or pressurized air – this means the cleaning process is quick, efficient and inexpensive.

The Selecta Spray Gun features a new type of internal flow control mechanism.

It permits the smooth and simple adjustment of the water or air flow across the entire range of the trigger movement. The desired maximum water or air flow can be set by using the small knurled screw located behind the trigger. It is then regulated via the trigger up to the set maximum value.

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THE SELECTA SPRAY GUN WITH CONUS AND WITHOUT TUBING CONNECTOR

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THE PRACTICAL SELECTA TABLETOP ROSETTE

The tabletop rosette, a threaded hollow cylinder, facilitates the installation of the Selecta Spray Gun in the sink or a countertop. The spray gun is then within easy reach. The tubing with the respective connector to the water or air supply is stored away underneath the counter or sink, thus keeping the work area clutter-free.

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The complete Selecta Spray Gun Set consists of a spray gun, tubing, water connector, attachments with storage rack and mounting hardware. It is available in the following versions:

Anschluss Connector	Standard-Set 8 Ansätze ① – ⑧ Standard set 8 attachments ① – ⑧
1/2"	80-10-80
3/4"	80-20-81

Anschluss lose beigelegt
Connector included in packing

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A SET OF ATTACHMENTS THAT MEETS EVERY NEED

Rinsing, suction or drying in every area of application is made possible through the set of 8 attachments.

The standard set of 8 attachments is delivered on a stable storage rack for mounting on the wall. All attachments are kept within easy reach.

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Attachment:

- for catheters, valves etc.*
- for drainage tubings*
- spray jet for rinsing out glassware etc.*
- for syringes and cannulae with Record cone*
- for measuring and blood pipettes*
- for syringes and cannulae with Luer cone*
- for bottles and Erlenmeyer flasks*
- water jet pump for suction cleaning*

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The Selecta Spray Gun features a new type of internal flow control mechanism. It permits the smooth and simple adjustment of the water or air flow across the entire range of the trigger movement. The desired maximum flow can be set by using the small knurled screw located behind the trigger. It is then regulated via the trigger up to the set maximum value.

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All attachments are connected to the spray gun by seating them on the safety cone located at the spray gun tip. Even high pressure cannot disconnect them. The gun grip is heat-insulated.

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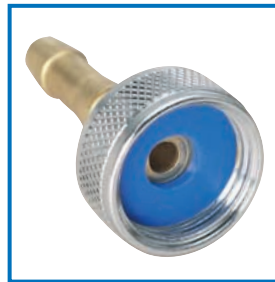
The Selecta Spray Gun can be ready for operation very quickly:
 1. *The storage rack with the attachments is fastened to a wall near the sink or work area.*

2. *The tubing is connected to the tap with a threaded connector (sizes: 3/4", 1/2"). If the spray gun is to be operated with compressed air, the tubing must be connected to the air supply by means of a suitable connector (can not be supplied by Stöckert Instrumente GmbH).*

Anschluss 1/2
Connector 1/2

Anschluss 3/4
Connector 3/4

Anschluss / Connector	Art. Nr. / Part No.
1/2"	80-10-15
3/4"	80-10-14

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STÖCKERT
 INSTRUMENTE GMBH

Stöckert Instrumente GmbH
 Lindberghstraße 25 · D - 80939 München
 Telefon: +49 89 323 01 - 0
 Telefax: +49 89 323 01 - 444
 E-Mail: info@stoekert.de
 http://www.stoekert.de