

marClamp[®] Cut IQ

Unbeatable for Efficiency and Economy



"Reuse makes economic sense. We value safety, reliability, and cost-effectiveness!"

marClamp[®] Cut IQ The new combination instrument for open surgical vessel sealing

SealSafe® IQ

The bipolar vessel sealing system, which consists of the maxium[®] high-frequency generator with its SealSafe[®] current and the bipolar marClamp[®] instruments, allows you to seal vessels and tissue bundles effectively and lastingly without any need for previous dissection or detailed exposure of the tissue to be sealed.

The SealSafe[®] IQ is a process-controlled current mode in which a continuous exchange of information takes place between the HF generator maxium[®] and the IQ instrument and in which the impedance of the tissue to be sealed is continuously monitored.



Thanks to the precise SealSafe[®] current specially adapted to this kind of application, only the tissue located between the instrument's jaws is sealed – lateral thermal tissue damage can thus be restricted to an absolute minimum.

The SealSafe® IQ current mode guarantees minimal tissue adhesion and prevents carbonization of the tissue when used correctly.

marClamp® Cut IQ

As well as for tissue preparation and vessel sealing, the marClamp[®] Cut IQ instruments are also used for subsequent tissue separation – with no need to change instruments!

The switch integrated in the instrument automatically triggers the special SealSafe[®] IQ sealing current once the tissue has been gripped, so no additional footswitch is required.

When sealing has been performed, the current is stopped and an acoustic signal sounds.



marClamp[°] Cut IQ Instruments and **maXium**[°] – the sealing system for fast and safe working

The reusable sealing system for lasting sealing of veins, arteries and tissue bundles with subsequent separation – and absolutely no need to change instruments!



Pressure (p) plus electric current (I) achieve the sealing

Thanks to the effective combination of high pressure (p) and electric current (I), a permanently sealed zone is reliably created on arteries, veins or tissue bundles. It is pivotal here that the pressure is maintained at a constant level throughout the duration of the application. This is ensured via the integrated switch, which only activates the current when the instrument is in the closed position.

During the sealing process, endogenous structural proteins such as collagen and elastin are transformed so that a permanently sealed zone is created as a result.

With this sealing method there is absolutely no need for foreign materials such as sutures or clips. The advantage of this method is the reduction in post-operative pain because no changes in tissue structures occur, as is the case e.g. when tissue is constricted by suture material.

Plug and Play – automatic identification of instruments with intelligent IQ

With hardware version 06 the name of the connected instrument is automatically displayed by maxium[®] as soon as an IQ instrument is plugged in, and the current associated with the instrument is set to a matching default value – it simply could not be more straightforward!

If you do not wish to work with the instrument-specific default values, simply change them as required and save the new values in the usual, program-specific manner. The advantage of this method is that when the IQ instrument is plugged in again, your customized values will appear instead of the default values.

The following items are currently available with IQ functionality together with the SealSafe® IQ current mode:

From software version V3.400 or higher: marClamp[®] IQ, marSeal IQ, marSeal Slim IQ and marSeal5 IQ

From software version V3.412 or higher: marClamp[®] Cut IQ.



Displaying the name of the connected instrument



Use of the two bipolar connecting sockets Automatic assignment of the required current type and set-up of the required performance parameters

Multi-use tools instead of disposables: more ways for everyone to take responsibility

Sequence of steps in the process:



Gripping the tissue

Automatic activation


Sealing the tissue

Cutting the tissue


All the benefits at a glance

- Reliable preparation, sealing and separation with just one instrument
- Reproducible and fast sealing process for vessels up to 7 mm with a burst pressure of over 360 mm Hg
- Minimal lateral heat spread of less than 2 mm
- Optimal cutting results due to sterile disposable blade
- Manual activation
- "Plug & Play" function
- Shorter operating times and hospital stays
- Substantial cost saving due to reusability and low proportion accounted for by disposables
- Easy handling and reprocessing
- Autoclavable at 134°C, acc. to ISO 17665-1

Cost comparison

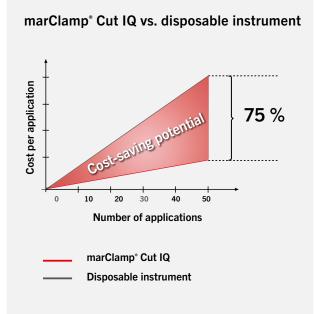



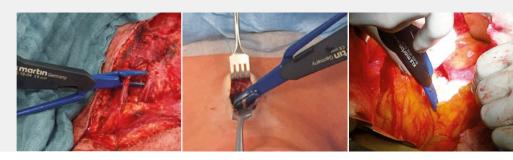

Thyroidectomy

Minimally-invasive video-assisted thyroidectomy (MIVAT)

Pancreatectomy

Applications

Gynaecology

- Lymphadenectomy
- Adhesiolysis
- Cystectomy

Visceral surgery

- Colectomy
- Gastrectomy
- Splenectomy
- Adrenalectomy

ENT

- Thyroidektomie
- Neckdissektion
- Laryngektomie

- Nephrectomy

- - - Partial cystectomy

Urology


Cystectomy

Hydrocele

Article overview

	marClamp [*] Cut IQ System Instrument, complete incl. cleaning tool and 3 cleaning brushes
80-632-18-04	marClamp [®] Cut IQ, 18 cm
	Components
80-632-00-04	Blade for marClamp® Cut IQ, sterile, pack of 10
80-630-58-04	Cleaning tool
80-632-01-04	Set of cleaning brushes, 3 pcs.



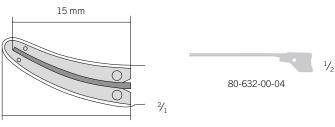

17 mm

KLS Martin Group

KLS Martin Australia Pty Ltd. Sydney · Australia Tel. +61 2 9439 5316 australia@klsmartin.com

Martin Italia S.r.l. Milan · Italy Tel. +39 039 605 67 31 italia@klsmartin.com

Martin Nederland/Marned B.V. Huizen · The Netherlands Tel. +31 35 523 45 38 nederland@klsmartin.com

KLS Martin UK Ltd. London · United Kingdom Tel. +44 1189 000 570 uk@klsmartin.com KLS Martin do Brasil Ltda. São Paulo · Brazil Tel. +55 11 3554 2299 brazil@klsmartin.com

Nippon Martin K.K. Tokyo · Japan Tel. +81 3 3814 1431 nippon@klsmartin.com

Gebrüder Martin GmbH & Co. KG Moscow · Russia Tel. +7 499 792-76-19 russia@klsmartin.com

KLS Martin LP Jacksonville - Florida, USA Tel. +1 904 641 77 46 usa@klsmartin.com KLS Martin Medical (Shanghai) International Trading Co. Ltd. Shanghai · China Tel. +86 21 5820 6251 china@klsmartin.com

KLS Martin SE Asia Sdn. Bhd. Penang · Malaysia Tel. +604 505 7838 malaysia@klsmartin.com

KLS Martin Taiwan Ltd. Taipei 106 · Taiwan Tel. +886 2 2325 3169 taiwan@klsmartin.com

kls martin

KLS Martin India Pvt Ltd. Chennai · India Tel. +91 44 66 442 300 india@klsmartin.com

KLS Martin de México S.A. de C.V. Mexico City · Mexico mexico@klsmartin.com

Gebrüder Martin GmbH & Co. KG Dubai · United Arab Emirates Tel. +971 4 454 16 55 middleeast@klsmartin.com

Do you know how to get all important information about the KLS Martin Energy Devices?

Please download KLS Martin App Energy Devices! The App is available for Android and iOS.



https://itunes.apple.com/de/app/ kls-martin-energy-devices/id1198171415?l=de&ls=1&mt=8



https://play.google.com/store/apps/ details?id=com.klsmartin.energydevices

Gebrüder Martin GmbH & Co. KG A company of the KLS Martin Group KLS Martin Platz 1 · 78532 Tuttlingen · Germany P.O. Box 60 · 78501 Tuttlingen · Germany Tel. +49 7461 706-0 · Fax +49 7461 706-193 info@klsmartin.com · www.klsmartin.com

90-279-02-07 · Rev. 01 · 2020-01 · Printed in Germany · Copyright by Gebrüder Martin GmbH & Co. KG · Alle Rechte vorbehalten · Technische Änderungen vorbehalten We reserve the right to make alterations · Cambios técnicos reservados · Sous réserve de modifications techniques · Ci riserviamo il diritto di modifiche tecniche