



## Inflatable unilateral models

Without ai	irways	
Size	Model	
4.5cm	RR450	
5.5cm	RR550	
7.5cm	RR750	
9.0cm	RR900	
With airwa	iys	
Size	Model	
5.5cm	RR551	
7.5cm	RR751	
Non-infla	table tampo	nade
Size	Model	
5.5cm	RR530°	


## Product usage directions

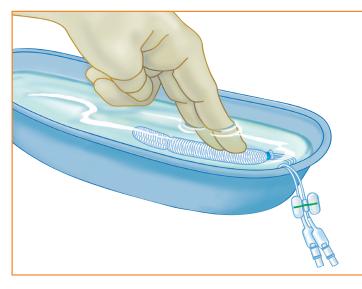

1 Soak in sterile water for 30 seconds.

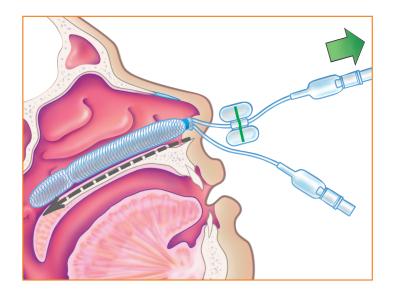


2 Insert along superior aspect of the hard palate until the blue indicator is past

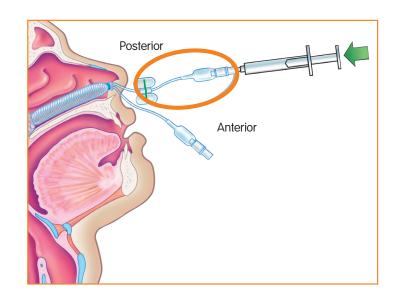


3 Inflate posterior balloon 1st (green line indicator) with air.


the nares.

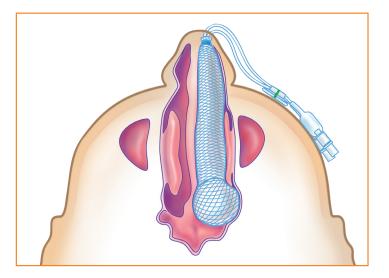

4 Inflate anterior balloon and ensure both pilot cuffs are firm.

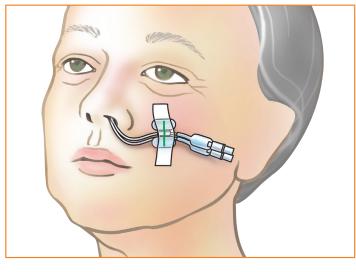

5 Discharge patient after a minimum of 15-20 minutes. Re-inflate to insure proper pressure and tape to patient's cheek.

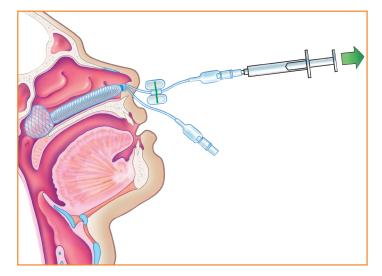

6 Removal should occur 24-72 hours after treatment.

NOTE: When there is sufficient pressure in the posterior balloon, remove the syringe from the inflation valve and gently pull the device anteriorly (as necessary) to ensure there is sufficient pressure on the posterior bleed site.

Please refer to the Instructions for Use (IFU) packaged with the product for a complete list of warnings, precautions and contraindications.

ArthroCare Corporation 7000 West William Cannon Drive Austin, TX 78735 USA

www.smith-nephew.com

Order Entry: 1-800-343-5717 Order Entry Fax: 1-888-994-2782

© 2015 Smith & Nephew, Inc. °Trademark of Smith & Nephew. Reg. US Pat. & TM Office.